

# cobas<sup>®</sup> HPV Test


## Sample color reference guide

Color reference guide for PreservCyt<sup>®</sup> (ThinPrep) and Surepath<sup>™</sup> Liquid Based Cytology (LBC) samples contaminated by blood.

The cobas<sup>®</sup> HPV Test has been validated for use with LBC contaminated with whole blood. Whole blood contamination in PreservCyt and Surepath samples showed no interference when present in concentrations up to 2% (v/v). Sample with visual appearance of blood contamination in excess of 2% may cause test interference.

For visual reference, the images below show primary PreservCyt and Surepath vials both fresh and aged after whole blood was added. The aged samples were stored for one week at ambient temperature.

### PreservCyt (ThinPrep) vials, fresh

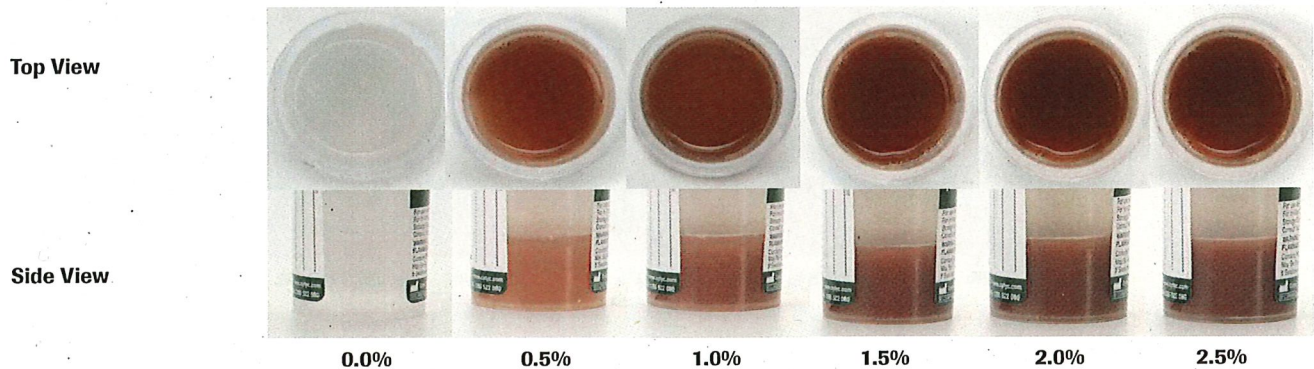

### PreservCyt (ThinPrep) vials, aged

